

PRECAUTIONS

For external use only • Do not use for purposes other than intended • Do not share your **OPTASE® Cooling Mask** • Do not apply to sensitive skin or open wounds • Do not ingest contents • Seek medical advice if symptoms persist or worsen • Inspect your **OPTASE® Cooling Mask** and discard it if damaged • Keep out of reach of children • The head straps in this product contain latex • Store at room temperature.

OPTASE®

Range of products



The **OPTASE®** Range (each sold separately).
For best advice on which products are suitable
for you, consult a medical professional
or speak directly to Scope.



UK

Unit 4 Amberley Court, Whitworth Road,
County Oak Way, Crawley, West Sussex, RH11 7XL.

Telephone: 0800 270 0253

Email: info@scopeeyecare.com

www.scopeeyecare.com

Ireland

Suite 5, Westland House, Westland Park,
Willow Road, Dublin 12, D12Y012, Ireland.

Telephone: 1800 816 005

Email: info@scopeeyecare.com

www.scopeeyecare.com



OPTASE® is a trademark of Scope Ophthalmics


Date of Issue: May 2023

OPTASE®

COOLING MASK ❄️



COOLING MASK x 1

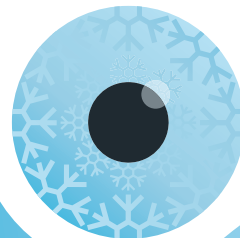

HYDROCOOL TECHNOLOGY

Cooling relief for puffy and inflamed eyes
and eyelids due to allergies.

Reusable cooling mask that relieves allergy
symptoms with soothing Cold Therapy Compress.

Soft & Comfortable

Adjustable

Cooling Allergy Relief

Reusable

SCOPE

OPTASE®

COOLING MASK ❄️



The **OPTASE® Cooling Mask** is a reusable cooling mask designed to relieve hayfever or allergic symptoms around the eyes, such as itchy, red, watery eyes as well as puffy or inflamed eyelids.

DIRECTIONS FOR USE

1



Place the **OPTASE® Cooling Mask** in a sealed plastic bag and place in the freezer for one hour.

2



Position the **OPTASE® Cooling Mask** around the head and close eyes for 10 minutes adjusting the strap for comfort.

Do not wear the **OPTASE® Cooling Mask** for longer than 20 minutes.

The **OPTASE® Cooling Mask** features HydroCool Technology to provide cooling therapy to closed eyelids.

RECOMMENDED USE

Cold therapies can reduce the symptoms of hayfever or allergic symptoms around the eyes, such as itchy, red, watery eyes as well as puffy or inflamed eyelids.

USEFUL INFORMATION

Contact lens wearers must remove lenses prior to using the **OPTASE® Cooling Mask**.

After treatment, it may be normal to experience some blurring in your vision. The blurriness should clear within a few minutes. If it does not clear within an hour, consult your eye care specialist.

Wipe immediately if exposed to food, oils, grease or other dirt. Air dry until mask is completely dry before next use.