

# OPTASE<sup>®</sup>

## DRY EYE INTENSE DROPS

**Instructions for Use – please read carefully**  
**OPTASE<sup>®</sup> Dry Eye INTENSE Drops, solution**  
**in single-dose containers**


### **What this is and what it is intended for?**

OPTASE<sup>®</sup> Dry Eye **INTENSE** Drops relieves symptoms of dry eyes such as irritation and burning sensations. This may also be used as a protectant against further irritation from dry eyes.

OPTASE<sup>®</sup> Dry Eye **INTENSE** Drops contains Glycerin. Glycerin is a humectant (moistening effect) that comes from plant sources. When applied topically to the eye, Glycerin protects and lubricates to relieve dryness and irritation. OPTASE<sup>®</sup> Dry Eye **INTENSE** Drops contains no alcohol, no preservatives, no ingredients of animal origin and are contact lens compatible.

### **Instructions for use**

1. Please wash your hands before use.
2. Open the aluminium pouch and pull out the strip of single-dose containers.
3. Remove one of the single-dose containers from the strip (Fig. 1, below).
4. Place the remaining single-dose containers back and close the pouch by folding the edge over.
5. To open, twist and pull tab to remove. Do not touch the tip after opening the container (Fig. 2, below).
6. Lean your head back (Fig. 3, below).
7. Pull down your lower eyelid with your finger and hold the single-dose container in your other hand. Squeeze the container so that one drop falls into the eye (Fig. 4, below).
8. Close your eyes and press the tip of one finger against the inner corner of the eye for around 1-2 minutes. This will prevent the drop running through the tear duct into your throat and most of the drop will remain in the eye (Fig. 5, below). If necessary repeat steps 6 to 8 with your other eye.
9. Discard the single-dose container after use.

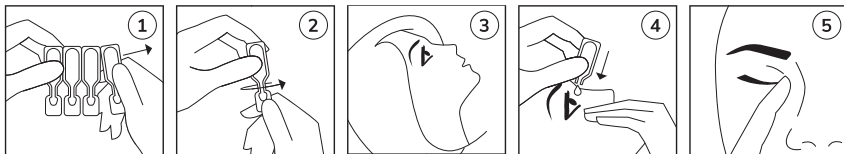

# OPTASE®

## DRY EYE INTENSE DROPS

### Precautions and warnings

- For external use only.
- To avoid contamination, do not touch tip of container to any surface.
- If solution changes color or becomes cloudy, do not use.
- Do not use if you are sensitive to any of the ingredients.
- After usage of one single-dose container, the remaining liquid must be discarded, it is not intended for later usage. Do not use a damaged or punctured single-dose container.
- The container is for single person use only.
- Do not touch the eye or contact lens with the tip of the single-dose container.
- Do not touch the tip of the single-dose container with fingers.
- Do not use after the expiry date shown on the pack.
- Wait at least 15 minutes between possible applications of other ophthalmic medication.
- Keep out of the reach and sight of children.

- OPTASE® Dry Eye **INTENSE** Drops should not be stored above 25°C/77°F, and should be protected from direct sunshine and heat.
- Discard the empty single-dose container appropriately.
- Stop use and ask doctor for advice if you feel pain, changes in vision occur, redness or irritation of the eye or if condition worsens or lasts more than 72 hours.
- Do not use if you are sensitive to any of the ingredients. If in doubt, consult a healthcare professional.

### Composition

Active Ingredient: Glycerin 0.2%

Inactive ingredients: Hydrochloric Acid, Sodium Hyaluronate, Sorbitol, Trometamol, Water.

**This revision:** June 2023

The OPTASE® brand and Scope logo are trademarks of Scope Ophthalmics Ltd.

# SCOPE

Scope Health Inc.  
79 Madison Av, 27 E 28th St. New  
York, NY, 10016, USA

Phone: +1 917 795 4227  
[www.optase.com](http://www.optase.com)  
[hello@optase.com](mailto:hello@optase.com)

OIS-101PL-1



CODE 39

65NOTXXXX/Y

FRONT SIDE

CODE 39



CODE 39

CODE 39

65NOTXXXX/Y

REVERSE SIDE

65NOTYYYY/X  
N° POSE