

**Precautions:** Keep out of reach of children • Do not apply directly to the eye • Do not use if you have known allergies to any of the ingredients • For external use only • Discontinue use if you experience any redness, itching or irritation and consult your Eye Care Specialist • Single use only • Suitable for Children aged 3 and older, and Adults



The OPTASE® family of products are designed to deliver comfort and soothing relief.



#### UK


Unit 4 Amberley Court,  
County Oak Way, Crawley, England,  
RH11 7XL.

**Telephone:** 0808 175 3200  
**Email:** [info@scopeeyecare.com](mailto:info@scopeeyecare.com)  
**www.scopeeyecare.com**

#### Ireland

Suite 5, Westland House, Westland Park,  
Willow Road, Dublin 12,  
D12Y012, Ireland.


**Telephone:** 1800 816 005  
**Email:** [info@scopeeyecare.com](mailto:info@scopeeyecare.com)  
**www.scopeeyecare.com**

OPTASE® and  are trademarks of Scope Ophthalmics.

# OPTASE®

## TEA TREE OIL EYELID WIPES


BIODEGRADABLE

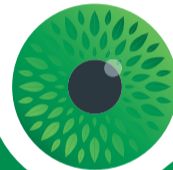

**PRESERVATIVE & SULPHATE FREE**

CLEANS | SOOTHES | REFRESHES  
Suitable for sensitive eyes & eyelids.



 Tea Tree Oil (TTO)

 Hyaluronic Acid

 Aloe Vera

 Chamomile

SCOPE

## What is in OPTASE® Tea Tree Oil Eyelid Wipes?

OPTASE® Tea Tree Oil Eyelid Wipes contain Tea Tree Oil (TTO) & Hyaluronic Acid as well as other ingredients to create a unique and gentle preservative free formulation.

## What is OPTASE® Tea Tree Oil Eyelid Wipes used for?

OPTASE® Tea Tree Oil Eyelid Wipes are used as part of a daily eyelid regimen.

Hyaluronic Acid is a natural substance found in many organs of the human body including the eye. Its special feature is that it moisturises and soothes the eyelids and can help improve eyelid elasticity.

Tea Tree Oil (TTO) is a natural essential oil from the Australian native plant Melaleuca Alternifolia. Our concentration of TTO was chosen to provide the optimum level of comfort.

OPTASE® Tea Tree Oil Eyelid Wipes are preservative free making them ideal for daily eye hygiene.

## Directions for use

		
1. Wash hands before use	2. Remove wipe from sachet and wrap over finger	3. Close eye and wipe gently and thoroughly across the eyelid and lashes several times in a side to side motion

		
4. Using a fresh wipe, repeat the process for the other eye	5. Discard the wipe after each use	6. No need to rinse after use